



Radiant Impressions Custom Breast Prosthesis

She will love her restored sense of wholeness

- Mirror image of remaining breast or patient selected size for bilaterals
- Lightweight, unique patented silicone foam core
- Durable construction - can be worn while swimming, running and during other physical activities
- Able to be worn in a non-pocketed bra
- Can adhere to chest wall
- Available in 30+ skin tone colors

Indications: Post breast surgery after a mastectomy, lumpectomy or reconstruction*; excessive keloid formation, bone loss, lymph node removal; large, hard to fit unilateral or bilateral mastectomy resulting in asymmetrical chest wall; ideal for lymphedema patients

**Reconstruction patients benefit from custom overlays.*



SELECTION


Model No.	Description
599S	Radiant Impressions Custom Breast Prosthesis (Scan)

Scanning Technology

Scanning appointment can be completed in 20 to 30 minutes – convenient for practitioner and patient.*

5 Easy Steps:



1. Confirm bra fit and select characteristics (skin tone color, freckles, veins, and nipple projection)
2. Prep patient with line markings
3. Take digital photos
4. Scan patient twice: a bra scan and a chest wall scan
5. Place reference markings on scan with mechanical or optical stylus

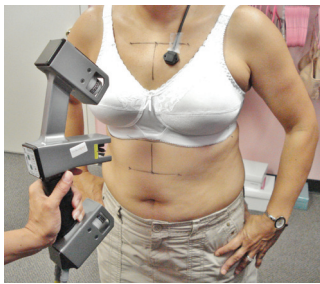


Scan patient twice



Scan 1: Bra scan

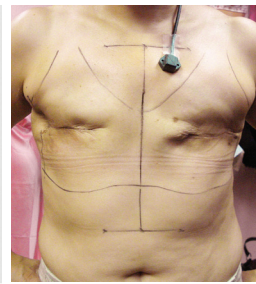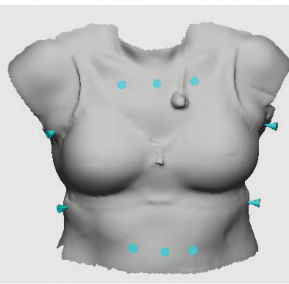

Scan 2: Chest wall scan

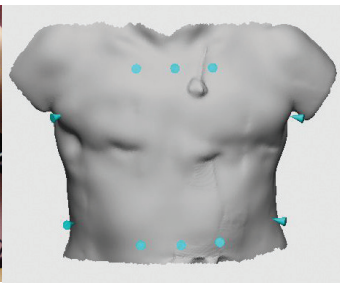

*Scanning protocol available for facilities with scanners or schedule a Fitting Event with a Trulife Scanning Technician.

Radiant Impressions Custom Breast Prosthesis

SELECTION

Model No.	Description
599S	Radiant Impressions Custom Breast Prosthesis (Scan)

ACCESSORIES AND REPLACEMENT PARTS

Model No.	Description
584	Color Swatches
592	Radiant Care Kit (Skin care products to clean, attach and care for Radiant Impressions custom breast prosthesis)
602	Radiant Impressions Surgeon's Sample Nipple Kit
570	Radiant Impressions Dealer Start-Up Kit (Sample prosthesis with display stand (choice of Caucasian or African American), sample overlay, semi-custom nipples, color swatches, Radiant Care Kit and literature)



584



592



602



570

 Non-Surgical Breast & Nipple Restoration
Radiant Impressions